



MERZ AESTHETICS

**NO FILTER.  
JUST HER.**

♥ 1207 likes

[www.merzaesthetics.com](http://www.merzaesthetics.com)

**BELOTERO® REVIVE**

# A SKIN PRESERVATION SOLUTION FOR YOUNG PATIENTS

In this changing age of insta beauty, young women need to look selfie-ready at all times. Living healthily, fitness classes and the latest food trends only go so far.

「 While beauty standards have never been higher 」

In social media especially it is important to look young and flawless. At the same time, being young means living young, which may lead to tired and dull skin.

「 Your next request might well be from a young patient who does not want a drastic transformation 」

Instead, they might request a natural-looking boost to their skin's firmness and elasticity, as they fear a change that might look fake. More than anything, they want to take care of their skin, and save its luster from fading.

Prevention and early intervention is key.



# REVIVE THEIR DREAM FOR YOUTHFUL SKIN

You know the importance of moisture to skin, even among younger patients. And yet their way of living and natural effects of aging both work to wear their skin down. What is needed is a solution that is not drastic but still effective.

「 BELOTERO® Revive does all that 」

- Improves skin hydration<sup>1</sup>
- Redefines skin texture<sup>1</sup>
  - Boosts skin quality<sup>1</sup>
- Fills depressions due to facial atrophic scars<sup>4</sup>
- Smoothens the skin<sup>1</sup>
- Reduces redness<sup>1</sup>

Hydration is a key feature of a healthy skin. BELOTERO® Revive improves skin moisturisation thanks to the glycerol in it and its hygroscopic properties.<sup>4</sup> Glycerol is an effective humectant, as it promotes water absorption from the dermis to the stratum corneum – providing moisture from the inside out.<sup>2,3</sup>


The other key ingredient, hyaluronic acid, contributes to the filler's quality while giving the skin a more radiant, young appearance.<sup>5</sup> Only BELOTERO® Revive currently offers the unique combination of these two ingredients.<sup>4</sup>

「 Revive: A UNIQUE COMBINATION of cross-linked HA and Glycerol<sup>4</sup> 」

NEW



	BELOTERO® Revive
Cross-linked HA contents	20mg/ml
Glycerol contents	17.5mg/ml
Phosphate buffer pH 7 q.s.	1.0ml
Needle	2x 30 G ½ Needle
Volume of gel per syringe	1.0ml

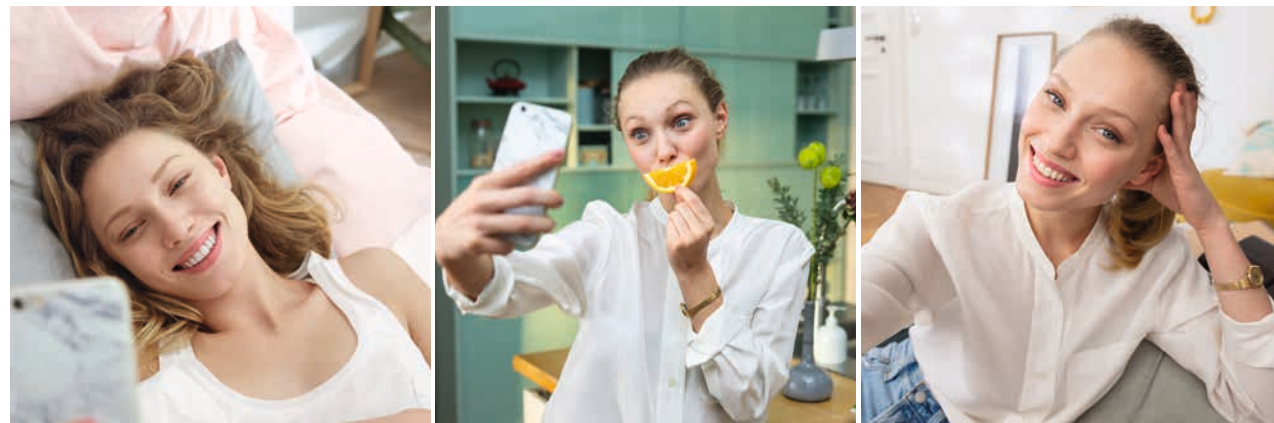

## INJECTING BELOTERO® REVIVE:

### 「 The technique 」

Recommended injection techniques are linear or serial threading, fanning, cross-hatching or serial puncture. BELOTERO® Revive can be used for all Fitzpatrick skin types. The injected quantity depends on the area to be corrected.<sup>4</sup>


### 「 Injection depth & sessions 」

#### Superficial dermis:

Facial atrophic scars, photo-damaged skin and fine facial lines in one treatment session.<sup>4</sup>

#### Mid to deep dermis:

Improvement of the elasticity or firmness of skin in three sessions four weeks apart.<sup>4</sup>

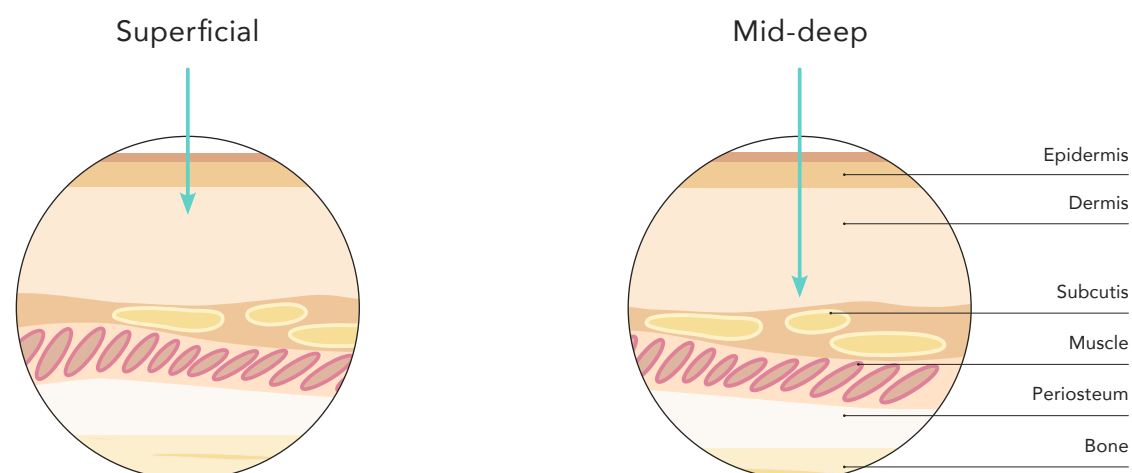

## THE TECHNOLOGY THAT REVIVES THEIR SKIN


### 「 Cohesive Polydensified Matrix 」

BELOTERO® Revive is defined by its CPM technology, as it gives the HA filler the unique rheological qualities that influence how it behaves during and after injection.


The BELOTERO® range relies on dynamic cross-linking technology (DCLT) including two additional phases that balance the elasticity, cohesivity and plasticity – tailored to each targetted landmark and depth. This enables BELOTERO® fillers to achieve treatment goals and optimise tissue integration.<sup>6</sup>




### 「 Standard cross-linking 」

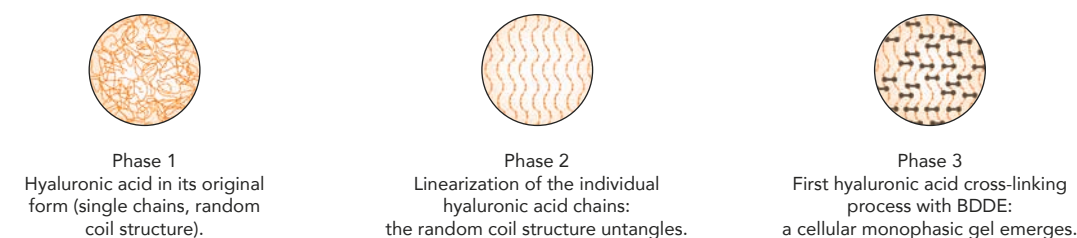

### 「 Patented dynamic cross-linking technology (DCLT)<sup>7</sup> 」

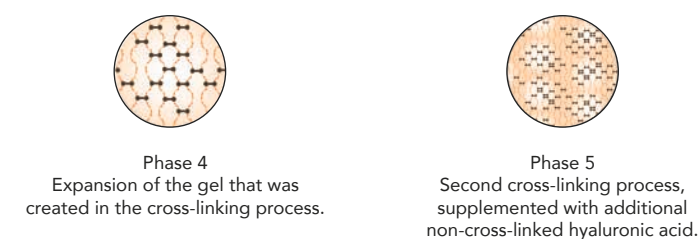

### 「 Multiple density zones – how product behaves in skin after injection<sup>6</sup> 」

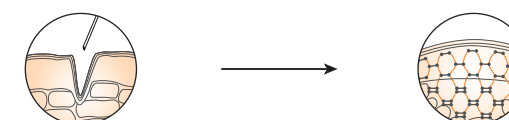



# REVIVE RESULTS: NUMBERS AND PICTURES

The Merz BELOVE study shows how BELOTERO® Revive improves skin elasticity, has an excellent safety profile, and a generally lower level of injection site pain. What else?

「 Up to 36 weeks 」

It improved skin hydration and reduced skin redness<sup>1</sup>

「 Over 80% 」

Of subjects were satisfied with the treatment at week 28<sup>1</sup>

「 90% 」


Would recommend the product to friends<sup>1</sup>

「 100% 」


All subjects showed an improved aesthetic outcome<sup>1</sup>




「 skin hydration 」

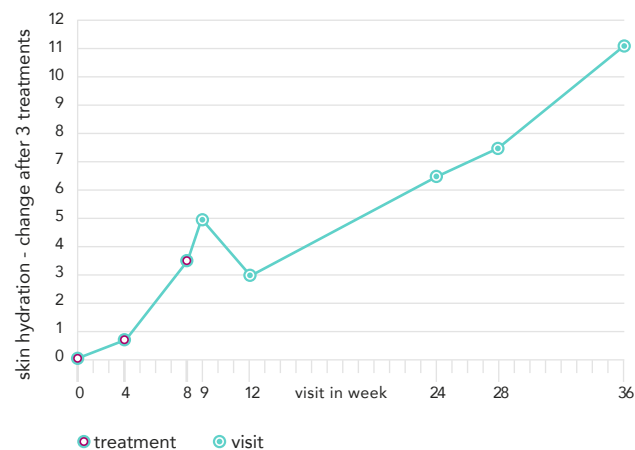

「 skin redness 」

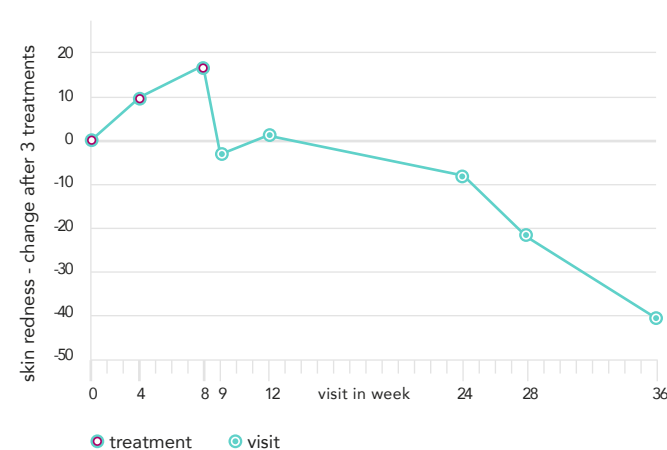

# THE SKIN BEFORE & AFTER BEING REVIVED

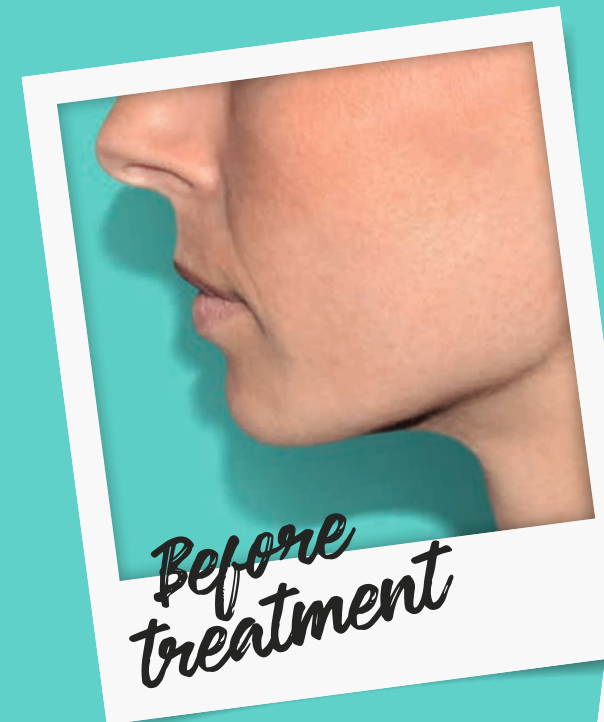

Patient treated with BELOTERO® Revive.  
Individual results may vary.

# PERFECT YOUR PRACTICE, WITH MERZ

As creator of BELOTERO® Revive, Merz has enjoyed long-standing success in the field of skin aesthetics.

Learn more about our mission and full range of products:

## 「 The BELOTERO® Range 」

Includes a wider selection of fillers which can correct imbalances and facial volume loss, define contours, lift, volumize, fill lines and wrinkles.



## 「 Merz Aesthetics 」

To us, aesthetics is more than just improved looks.

We believe each patient is beautiful in their own way, and treat them likewise.

With over a century of experience, we firmly believe in innovation and trusted relationships to help you provide your patients with a unique treatment solution.

For further information please visit:

[www.merzaesthetics.com](http://www.merzaesthetics.com)

BELOTERO® is a registered trademark of Merz Pharma GmbH & Co. KGaA.

BELOTERO® is manufactured by Anteis S.A.  
Chemin des Aulx 18, 1228 Plan-les-Ouates,  
Geneva, Switzerland

Merz Pharmaceuticals GmbH,  
Eckenheimer Landstraße 100,  
60318 Frankfurt am Main, Germany  
[www.merzaesthetics.com](http://www.merzaesthetics.com)

<sup>1</sup> Merz BELOVE Study, 2019

<sup>2</sup> Fluhr JW et al. Br J Dermatol 2008; 159:23-34

<sup>3</sup> Schwartz J, Friedman AJ. J Drugs Dermatol 2016; 15:1289-94

<sup>4</sup> BELOTERO® Revive Instruction for use Version 3.0, 2018

<sup>5</sup> Belmontesi M. et al J Drugs Dermatol 2018; 17:83-8

<sup>6</sup> Tran C et al. Dermatology 2014; 228:47-54

<sup>7</sup> Patent family publication number WO2005 / 085329